



SterileLight™
SL-UVC-IDC-01

ILLUMICIDE™ Disinfecting Case

Quick Reference Guide



40 LED Array Provides
360° Disinfection



Disinfection Preserving Storage Mode
Constant Temperature Drying



Dual Rechargeable Batteries
Provide 60 Quick Disinfects



Intelligent Safety Design
Shuts Off Automatically When Opened

Overview

SterileLight's **ILLUMICIDE™** Disinfecting Case is a high-quality, professional-grade surface disinfection device.

Provides powerful surface disinfection in just three minutes with UV-C technology. Eliminate 99.9 percent of bacteria (or 99.99 with 2 cycles) within this portable case that doubles as storage. The 40 light, instant-on array is mercury-free, unlike competitor's outdated technology that unnecessarily harms the planet.

Perfect for disinfecting any object likely to carry infection such as: iPads, smartphones, wallets, gloves, keys, protective facemasks, protective eyewear, hearing protection, tape measures, CPAP gear, thermometers, stethoscopes, blood pressure cuffs, watches, eyeglasses, razors, toothbrushes, pens and pencils, calculators, baby bottles and pacifiers. etc.



You can preserve the disinfected condition of your items by storing them in the case with constant temperature humidity control. (Disinfect & Store)



ILLUMICIDE™ Disinfecting Case Quick Reference Guide

SterileLight™
SL-UVC-IDC-01

3 Operating Modes

1 Quick Disinfect (clean items prior to disinfecting)



Press for 3 Minute Quick Disinfection Cycle (180 seconds)
Battery or AC Power

2 Disinfect & Dry



Press for 3 Minute Disinfection Cycle (180 seconds) followed by a 30, 60, or 90 minute Smart Drying Cycle. Once complete, device enters Disinfect & Store mode.
AC Power Required

3 Disinfect & Store



Press for 3 Minute Disinfection Cycle (180 seconds) AND ongoing Disinfection Preservation Cycles.
AC Power Required

5 LED Counter




Displays the number of seconds or minutes remaining in the cycle for the selected function

7 Stainless Steel Tray



Supports items to be disinfected and raises them above the reflective base of the device.

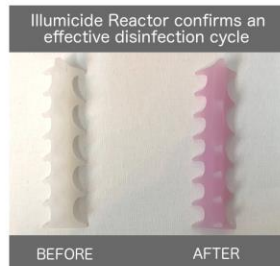

Note: Reactor will fade to white within 15 seconds. Not recommended for use during drying cycle as it may reduce reactor effectiveness.



NOTE: Upon receipt of the device, please charge overnight.

4 ON/OFF Button



Press to power the device ON or OFF

6 OPEN Button



Opens the device cover and initiates the auto shut-off safety feature.

8 Power Receptacle



USB-C transformer power receptacle. Use for charging device, Disinfect & Dry, and Disinfect & Store



Power receptacle is located in the lower center on the back of the device.



SterileLight™
SL-AC-IDC-01

If you added the Anti-Microbial Panel Protector...

Then congratulations on your purchase of SterileLight's CountDown Plus medical-grade Anti-microbial Panel Protector for your ILLUMICIDE Disinfecting Case. You've now joined a growing community that have the additional peace of mind gained by a germ and bacteria free surface on the frequently touched button panel.



Statue of Liberty Museum



SterileLight™
SL-AC-IDC-01

The Panel Protector is custom cut and is the same medical-grade anti-microbial film that's used in medical settings across the country, the same film used in our screen protectors for iPhones and iPads, and is the same film as our surface protection mats.



Visit the link below to view the Countdown Plus Video Installation Guide:

https://sterilelight.com/prp_sl-cd-plus/

PLEASE NOTE: You'll periodically see small air bubbles appear which are easily worked out when pressed outward with the fingers (just as screen protectors for your phone). This occurs naturally due to button presses and objects being placed on the surface.